


QUICKTIP

Rechargeable Hearing Aid StarLink Chargers 2.0

The StarLink Charger 2.0 operates as a charging station and case for standard and custom lithium-ion rechargeable hearing aids. The RIC RT, mRIC, and custom hearing aid styles have unique StarLink Charger 2.0 variants. Each charger must be plugged into an external power source (e.g., wall outlet) to function/charge the hearing aids. All chargers come with on-board turbo charging. Turbo charging automatically occurs when the hearing aid battery is low. The hearing aid will rapidly charge, providing approximately 3 hours of use time with only 10 minutes of charging.

NOTE: Each hearing aid ships with a unique charger depending on the style (RIC RT, mRIC, or Custom). The correct charger must be used to achieve proper charging. The underside of the charger will state which hearing aid style it is designed for. [Fig 1]

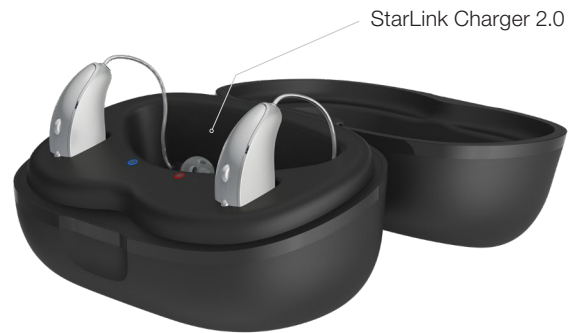

Model: mRIC R
 SN XXXXXXXX REF 761
 Starkey Laboratories, Inc.
 6700 Washington Ave. S
 Eden Prairie, MN 55344 USA
 FC CE

Figure 1

LED Indicators for Hearing Aid Charging

The inside of the chargers is equipped with two LED indicators to provide information about hearing aid charging.

- When the hearing aids are placed into the charger, the LED lights will illuminate to indicate successful placement.
- When the hearing aids are charging, the LED lights will blink.
- When the hearing aids are fully charged, the LED lights will be solid.



Use

- 1 Hearing aids will be fully charged in 3.5 hours.
- 2 It is safe to keep the hearing aids in the charger as they cannot overcharge.
- 3 Charging will occur with the lid open or closed.
- 4 Hearing aids automatically power on when removed from the charger and power off when inserted into the charger.

NOTE: *The hearing aids will power ON in the charging case if there is no power source.*